


A Small Product That Solves A BIG Problem

MEDIpoint SPLINTER OUT

- Sterile
- Individually Wrapped
- Disposable – Use Once and Discard
- Surgically Sharp & Easy to Grip
- Specially designed for Splinter Removal
- Packed in a Handy Plastic Case – Easy to Carry in a Backpack or Glove Compartment
- Registered with the FDA and Health Canada

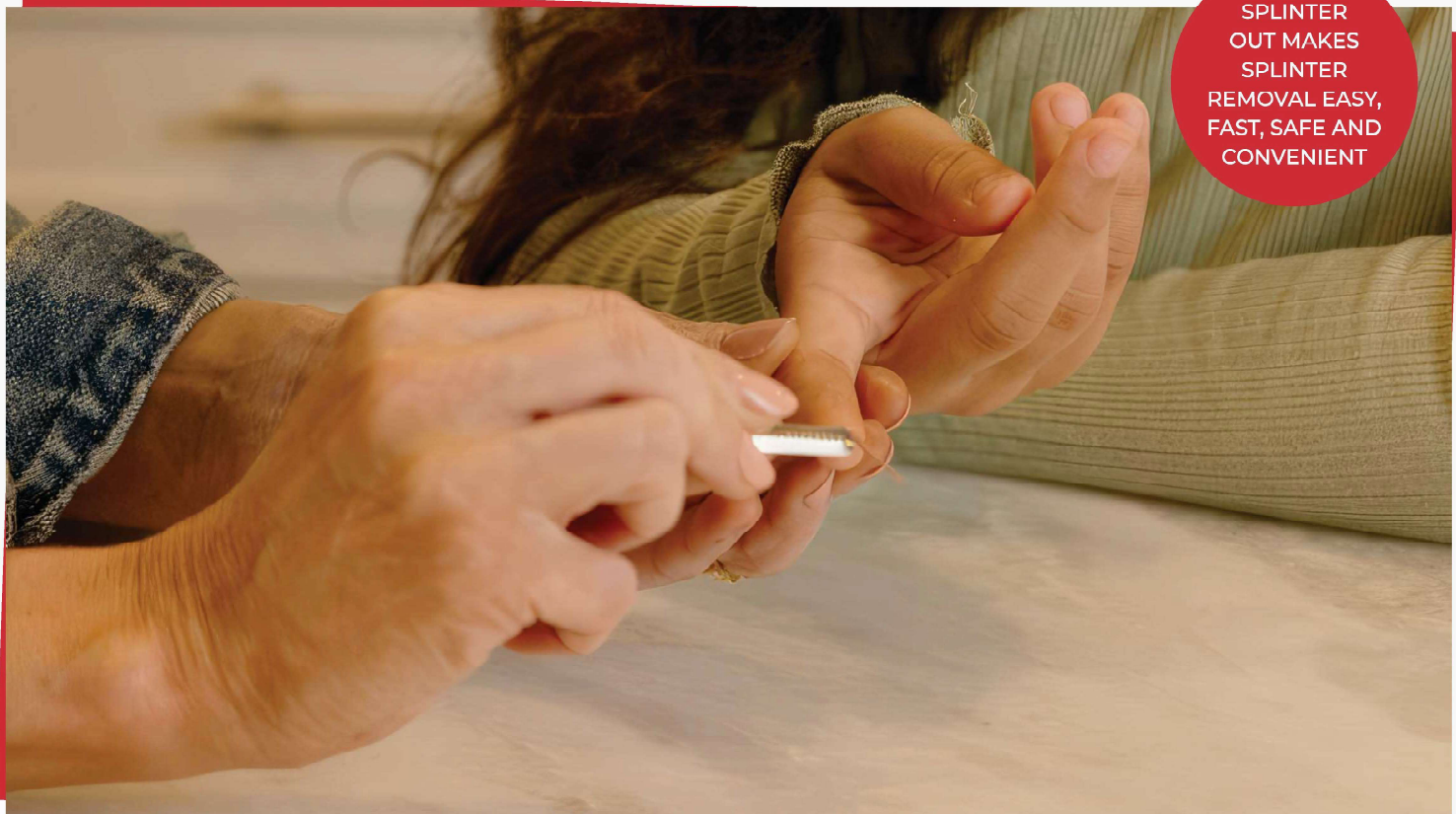

SPLINTER
OUT MAKES
SPLINTER
REMOVAL EASY,
FAST, SAFE AND
CONVENIENT

A Small Product That Solves A BIG Problem

MEDpoint

SPLINTER OUT

Instructions For Use

STEP 1



Wash hands thoroughly with soap and water.

STEP 2



Clean the area with alcohol pad and let dry.

STEP 3




Tear open the Splinter Out® pouch to expose the easy-grip end and remove from the package without touching the sharp tip.

STEP 4



Use the sterile point to gently loosen the skin above and around the splinter until the splinter can be lifted out. Do not remove splinter from its entry point. That will just drive the splinter deeper into the skin. Use the tweezers only once the splinter has cleared the skin surface.

STEP 5

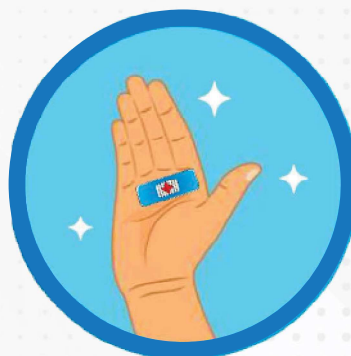

After removing the splinter with the Splinter Out®, bandage the area and keep clean until healed.

STEP 6



USE EACH SPLINTER OUT® ONLY ONCE, THEN DISCARD.

MEDpoint