



STEREO OPTICAL

To Whom It May Concern,

We have recently received questions about how best to clean and disinfect Stereo Optical vision screener surfaces without damage. Here are our suggestions:

Clean and disinfect the forehead activator and surrounding area gently with disinfectant wipes or a soft, slightly damp, and lint-free cloth with alcohol. We have used 70% isopropyl alcohol to clean and disinfect our equipment surfaces. Clean gently with a damp cloth and never spray alcohol directly on the screener surfaces.

Please feel free to contact us with any questions or for more information.

Best Regards,

Shirley Yin-Piazza, Sr. Marketing Manager
Stereo Optical Co., Inc.
8600 W. Catalpa Ave., Suite 703
Chicago, IL 60656-1116
Phone: (773) 867-0380
Fax: (773) 867-0388

STEREO OPTICAL COMPANY, INC.

8600 W. Catalpa Ave., Suite 703, Chicago, IL 60656-1116 U.S.A. Phone: 1-773-867-0380 1-800-344-9500 Fax: 1-773-867-0388

E-mail: sales@stereooptical.com Website: www.stereooptical.com